

Protective Face Shield - Assembly Instructions




SCAN ME

Scan QR code for link to YouTube video instructions

- Assemble in clean and sanitised conditions
- Additional PPE such as gloves and facemask recommended
- Face shield materials compatible with Ethanol, IPA and dilute chlorine solution



Protective Face Shields are supplied in kits

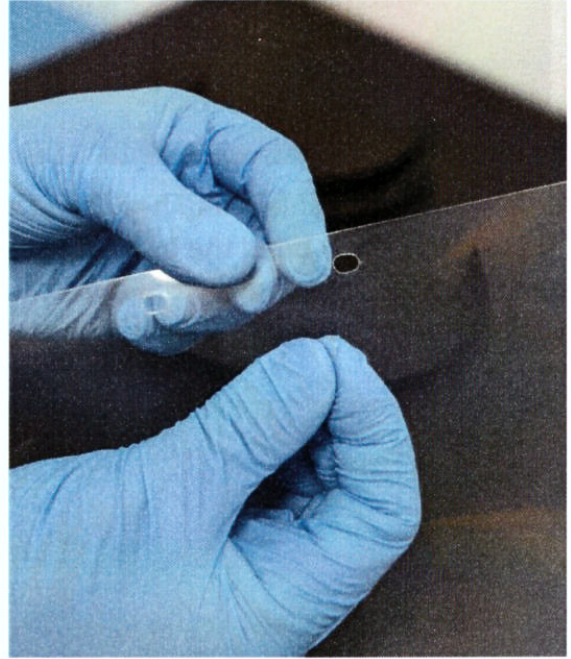

Remove and discard any hole pieces



Remove protective film from both sides of PET visor shield



Locate visor holes over two centre location lugs, hold in place



Holding the visor shield and headband in the centre, gently bend the ends of the headband forward and clip the outer visor holes over the lugs one at a time



Double over the ends of the elastic strap with the loose end on the inside



Align the holes in the elastic strap and stretch over the headband hook. Repeat on the opposite side. Adjust for comfort.



Disinfect your face shield before use and in accordance with your workplace procedures

TECHNICAL INFORMATION

Headband: Polypropylene
Visor Shield: PET 0.4mm thick
Elastic Strap: Polyester

MATERIAL COMPATIBILITY FOR DISINFECTION

Ethanol >78%
Isopropyl Alcohol >75%
Dilute Chlorine Solution
Disinfect in accordance with your workplace procedures