


FFP2 RESPIRATOR MASK

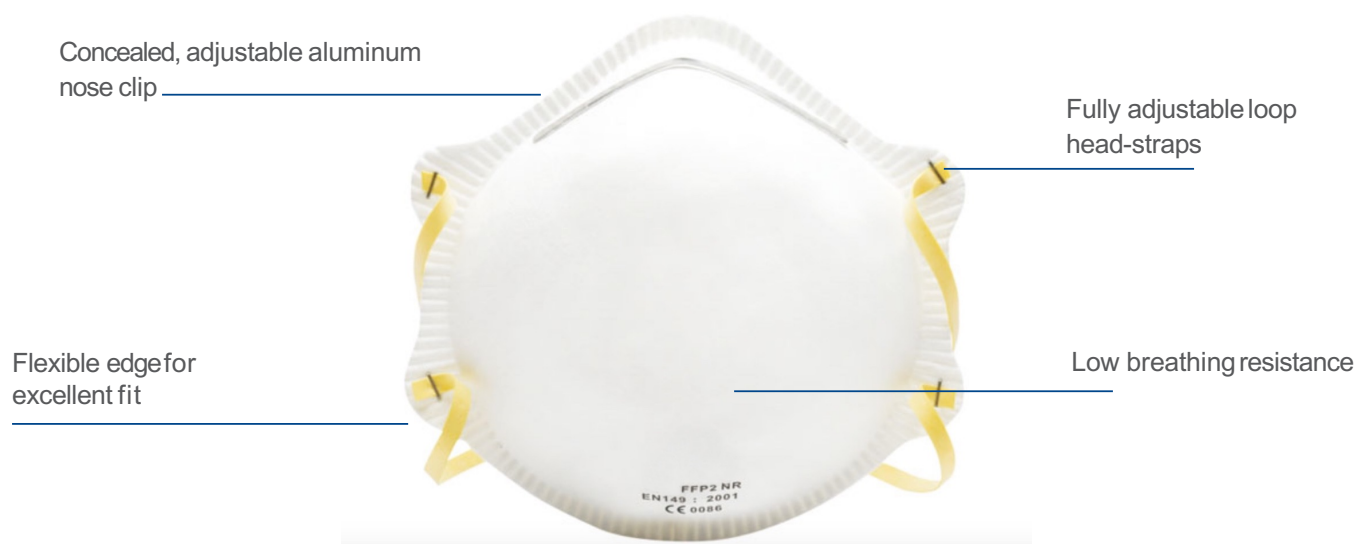

FFP2 Non-Valve Respirator Masks are a moulded style respirator mask that meets the EN 149 FFP2 standard. The mask is comfortable to use due to its flexible edges that can easily adapt to the face of the user. The inner and outer layers are manufactured from high quality polypropylene materials that are soft on the skin. The filter material of the mask provides high protection and low breathing resistance during use.

The respirator fits well with goggles or helmets and is convenient for different types of procedures in the ward.

Respirator Fitting Instructions



1. As with any respiratory device the wearer must be adequately trained prior to use. Before use, the wearer should always check that the mask is in good condition – no dirt, no damage of headstrap attachment, etc.
2. Hold the respirator in hand with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand.
3. Press the respirator firmly against your face with the nosepiece on the bridge of your nose.
4. Stretch and position the top band high on the back of the head. Stretch the bottom band over the head and position below your ears.
5. Using both hands, mold the metal nosepiece to the shape of your nose.
6. To test fit; a) cup both hands over the respirator being careful not to disturb the position, b) inhale vigorously. If air leaks around the edges, reposition the straps or adjust strap tension for better fit.
7. Remember, careful observance of these fitting instructions is an important step in safe respirator use.